



Control panel

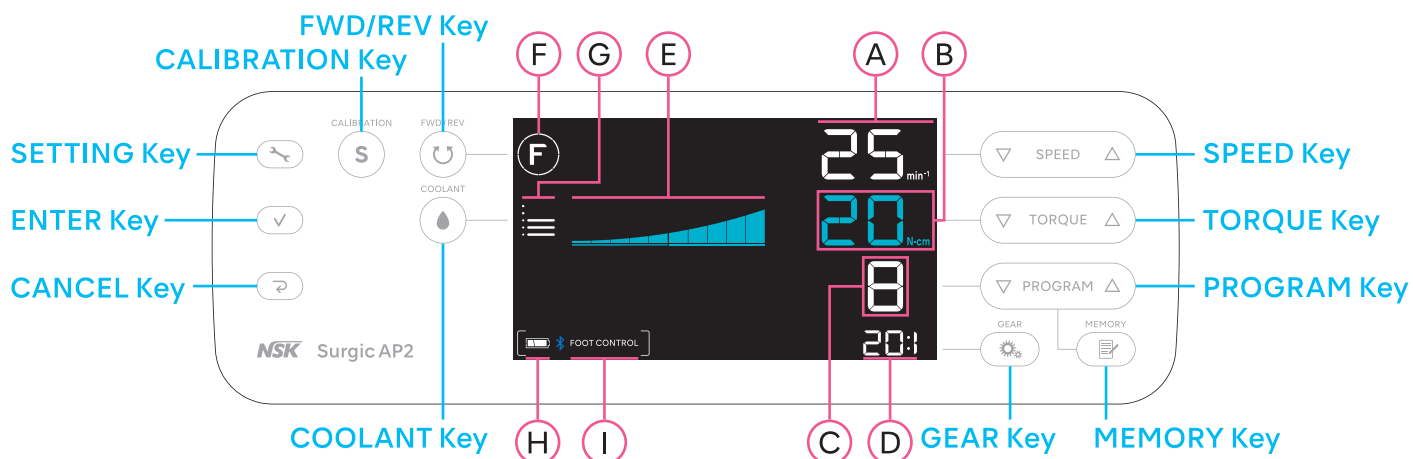

Foot control

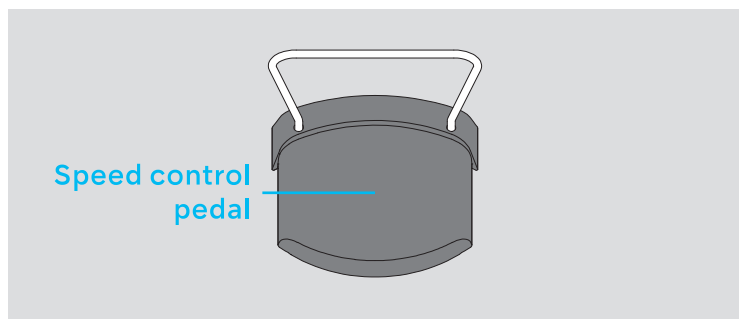


A	Rotation speed
B	Torque
C	Program No.
D	Gear ratio
E	Current operating torque level
F	Rotation direction
G	Coolant solution flow level
H	Battery level of the foot control
I	Status of connection between the foot control

Preparation for Use

- Connect the AC power cord.
- Remove the battery cover from the foot control.
- Insert three AA batteries.
- Attach the battery cover.
- Mount the coolant solution hanger post.
- Turn the pump cover clockwise 180°.
- Insert the irrigation tube.
- Turn the pump cover counterclockwise 180°.
- Insert the irrigation tube into a pack.
- Connect the motor cord.
- Connect the handpiece.
- Attach the tube holders.

To "Starting operation"


Starting operation

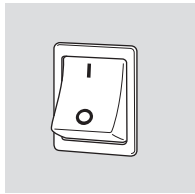


Turn on the power of Surgic AP2.



The above is displayed.

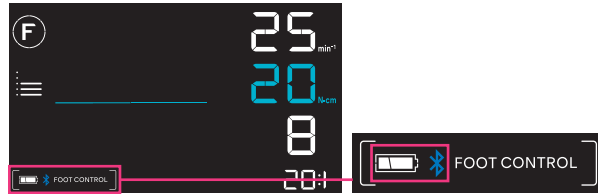


The normal screen is displayed. Make sure that the foot control is connected.

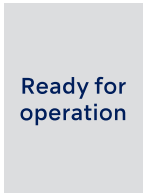

Ready for operation

Pairing with foot control

This operation is not necessary when using Surgic AP2 for the first time after purchase. If the foot control is unpaired or when using newly purchased foot control, conduct the following operation. NOTICE : Paring can be done within 10 minutes after inserting the battery.

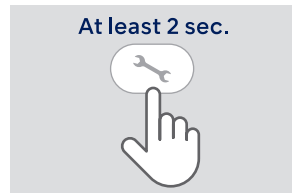


Press and hold the SETTING Key. "BLE SET" is displayed.




Press the ENTER Key.



The letters on the screen except "PAIRING" and "FC" flash on and off.

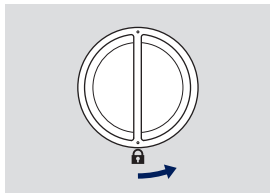

Remove the battery cover from the foot control.

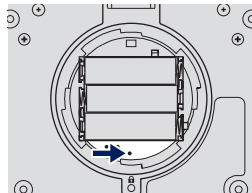

Press the button shown above with a pointed tool.



The control unit makes a beep and then "OK" will appear. Pairing is done. Make sure that the device is connected.



Press the CANCEL Key to return to the normal screen.